

Eclipse Foot



Product description

Eclipse Foot is a super absorbent dressing, designed to be applied securely around the foot area to manage wound exudate.

The dressing combines a rapid wicking polyester and viscose woven face with a sheet of highly absorbent crystals and mechanically bonded cellulose pad. The beige polyethylene backing is highly breathable, it provides a viral and bacteria proof barrier in addition to preventing strike-through.

Indications

Suitable for moderate to heavily exudating wounds, including: superficial wounds, pressure ulcers, arterial ulcers, diabetic foot ulcers.

Contraindications

- Arterial bleeds.
- Heavily bleeding wounds.

Method of use

Open out the dressing fully, white side up placing the end of foot in the middle of the dressing so the heel is at the edge. Fold the side panels over the foot one at a time and secure with tape. Eclipse Foot can be used under compression bandaging.

Mode of action

Eclipse Foot is a super absorbent wound exudate management dressing that is designed to handle high levels of exudate leading to fewer dressing changes and therefore reduced costs. The non-strike through, bacteria proof backing reduces infection rates. Eclipse Foot is highly breathable with a high moisture vapour transmission rate reducing the potential for maceration. The rapid lateral wicking central core allows for an even distribution of fluid across the dressing maintaining a low dressing profile.

Frequency of change

Wear time will depend on the level of exudate, daily changes may be required, but Eclipse Foot can be left in place for up to 7 days. Because of the excellent fluid handling capability of the dressing, it may become heavy and require changing.

Sizes/specification

33 cm × 48 cm. Available in packs of five.

Advancis Medical – www.advancis.co.uk

Tel: 01623 751 500 | Fax: 0871 264 8238 | Email: info@advancis.co.uk

Address: Lowmoor Business Park, Kirkby-in-Ashfield, Nottingham, NG17 7JZ