



## Product description

Medi Derma-S Barrier Film and Cream can be safely used for Wound care, Ostomy care and Contenance care to provide protection against harmful bodily fluids and adhesives without stinging. They are also ideal for protecting delicate areas when applying a sheath. The Medi Derma-S Range of Wound and Skin Care Products provides suitable protection against at-risk skin and helps maintain prevention of skin breakdown.

## Indications

- Can be applied to tender or broken skin without stinging.
- Provides protection around the area where the device is to be applied by forming a transparent coating.
- Protects skin from exudate and adhesives.
- No effect to the adhesion of the pouch or adhesive device.
- Only a small amount required for each application.
- Quickly absorbed into the skin using a gentle massage.
- Dries in seconds.
- Quick and easy to use.

## Contraindications

- Do not use on infected areas of skin.
- Only use as directed.
- Do not use these products if there are any signs of irritation and consult your clinician accordingly.

## Method of use

- Ensure that skin is clean and dry before applying around area where pouch or adhesive device will be applied.
- Wait for a moment until film or cream has dried.
- Apply pouch or adhesive device as normal.

## Overview

In recent case studies, the use of these barrier products has reduced erythema, excoriation, maceration, and pain in a range of cases involving skin damage or wounding. One case study also demonstrated a reduction in periwound breakdown. In all of these case studies, clinicians rated the Medi Derma-S products "as either as good as or better than the products they usually used".

## Sizes and formats available

Description	Pack size	Product code	PIP code
Medi Derma-S Medical barrier film (silicone-based, healthcare grade)	Aerosol (75 mL)	60291	341-3176
	Wipes (30 pack)	60307	341-3184
	Applicators (1 mL – 5 pack)	61076	362-8716
	Applicators (3 mL – 5 pack)	61090	362-8724
Medi Derma-S Barrier cream (healthcare grade)	Sachets (2g – 20 pack)	60338	341-3317
	Tube (90g)	60345	341-3325

For further information regarding the Medi Derma-S range, or to obtain samples, please contact Dave Kenton, National Sales Manager. Email: [dave.kenton@medicare-plus.com](mailto:dave.kenton@medicare-plus.com)  
Tel: 07880 032005

## Skin at presentation



## Skin following 2 days of treatment including Medi Derma-S barrier film

