

Advazorb Heel



Product description

Advazorb Heel is a conformable, low-adherent, hydrophilic, polyurethane foam dressing with a breathable film backing designed to fit around the heel.

It is a non-adherent foam dressing with excellent fluid handling properties, ideal for use under compression bandaging with its fluid retention, low-profile and low-friction film backing.

Indications

Suitable for acute and chronic wounds, including: cuts and abrasions, superficial burns, surgical wounds, foot ulcers, pressure ulcers and diabetic ulcers.

Contraindications

- Arterial bleeds.
- Heavily bleeding wounds.
- Vascular fungating tumours.

Method of use

Apply directly to the wound on the heel, pink side up and secure in place with tape, appropriate bandage or secondary dressing.

Mode of action

Advazorb Heel is a hydrophilic foam dressing that rapidly absorbs exudate and retains it within the dressing. The film backing provides a bacterial barrier and prevents strike-through. The dressing maintains a moist wound healing environment with a high moisture vapour transmission rate. It is soft and conformable for increased patient comfort.

Frequency of change

The product can be left in place for up to 7 days, but should be changed immediately if strike-through is apparent or when the dressing reaches capacity. If exudate is visible around the edges of the dressing, this is a clear indication that a dressing change is required. Clinical observation is necessary to determine the required frequency of change.

Sizes/specification

17 cm × 21 cm. Available in packs of five.

Advancis Medical – www.advancis.co.uk

Tel: 01623 751 500 | Fax: 0871 264 8238 | Email: info@advancis.co.uk

Address: Lowmoor Business Park, Kirkby-in-Ashfield, Nottingham, NG17 7JZ