ELECTTM Superabsorber





Product description

ELECTTM Superabsorber wound dressing is indicated for the treatment of heavily exuding wounds. It is composed of a sodium polyacrylate superabsorbent cellulose core, a non-adherent polyethylene wound contact layer, and a protective blue strikethrough-resistant top layer.

Indications

ELECTTM Superabsorber is indicated for the treatment of heavily exuding wounds, such as pressure sores, arterial ulcers, venous leg ulcers, diabetic ulcers, post-operative wounds healing by second intention, laparotomy wounds, fistulae, and on superficial second-degree burns.

Contraindications

- Known sensitivity to the product itself or to its components.
- Do not use in tunnelling wound pockets as the product can expand considerably when wound exudate is absorbed.
- Do not use on lightly exuding wounds as the wound may dry out and suction pain may occur.

Method of use

Prior to applying a new dressing, cleanse the wound and pat dry the wound edges. The size of dressing depends on the wound size; the dressing should overlap the wound by approximately 2 cm-3 cm. Apply the white, non-adherent wound contact layer to the wound surface. Use a suitable material for dressing retention.

Mode of action

The dressing is highly absorbent^{1,2} and will bind large volumes of exudate within its superabsorbent core³. The extensible polyethylene wound contact layer prevents adhesion to the wound⁴, while the protective blue top layer protects patients' clothing and bedding against strikethrough and leakage2.

Frequency of change

Dressing change frequency should be determined by the clinician in charge, based on the condition of the wound (e.g. the level of exudate).

Sizes/specification

 $10 \,\mathrm{cm} \times 10 \,\mathrm{cm}$, $10 \,\mathrm{cm} \times 20 \,\mathrm{cm}$, $20 \,\mathrm{cm} \times 20 \,\mathrm{cm}$, $20 \,\mathrm{cm} \times 30 \,\mathrm{cm}$. Available in packs of 10.

References (in vitro)

- 1) DOF DS/12/278/R
- 2) DOF DS/12/295/R
- 3) DOF DS/12/279/R
- 4) RD/12/071

For more information, please contact the Smith & Nephew customer care team on 01482 673848 or visit our website at www.smith-nephew.com

Asset no. 43107

